

3M

Littmann® Stethoscopes



The first test of **commitment**  
to quality patient care.

9-10



Acoustic  
Performance

### Cardiology

Designed to meet the needs of demanding clinicians. Provides outstanding acoustic sensitivity to help you perform detailed diagnostic auscultation.

Millions of medical professionals around the world begin their medical education with the purchase of a 3M™ Littmann® Stethoscope. The outstanding acoustic performance and high quality of their Littmann stethoscopes help these professionals express their dedication from the first day of school throughout their entire careers.

#### 3M™ Littmann® Cardiology III™ Stethoscope

is one of the most recognized stethoscopes in the profession, featuring tunable diaphragms on both sides to deliver outstanding acoustic performance, diagnostic versatility, and convenience for treating adult and pediatric patients. The pediatric side of the chestpiece converts to a traditional open bell by removing the tunable diaphragm and replacing it with the included non-chill bell sleeve.


#### 3M™ Littmann® Master Cardiology™ Stethoscope

features tunable diaphragm technology—a 3M invention—that lets clinicians hear different frequency sounds by simply adjusting the pressure on the chestpiece. Unlike two-sided models, there's no need to turn over and reposition the chestpiece. Hold the chestpiece with light pressure to hear low frequency sounds; press a little more to hear higher frequency sounds. This time-saving feature allows clinicians to focus on the patient, while the non-chill rim provides patient comfort.

3M™ Littmann®  
Cardiology III™  
Stethoscope



3M™ Littmann®  
Master Cardiology™  
Stethoscope

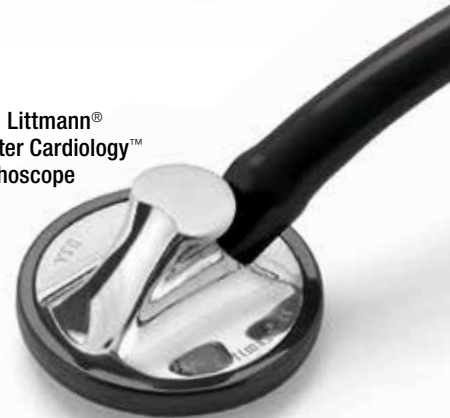



# Uniquely Yours

## Littmann cardiology stethoscopes offer:

### 1. TUNABLE DIAPHRAGM TECHNOLOGY.

A 3M invention that lets you hear different frequency sounds without turning over and repositioning the chestpiece. Hold the chestpiece with light pressure to hear low frequency sounds; press a little more to hear higher frequency sounds.

### 2. PRECISION CHESTPIECES.

Chestpieces are designed to provide excellent sound quality over a lifetime of use. Chestpiece materials are selected for their excellent acoustic properties, plus the strength to resist the impacts, scratches, and chemicals of daily use.

### 3. HIGH QUALITY, DUAL-LUMEN TUBING.

Next-generation tubing with improved resistance to skin oils and alcohol for longer tubing life. Less likely to pick up stains. Cardiology Series models feature two sound paths in one tube which eliminates the rubbing noise of traditional double tubes.

### 4. ADJUSTABLE HEADSET.

Headset tension is easily adjusted for head size and comfort by squeezing together or pulling apart the eartubes. Angled eartubes align with ear canals for comfort and maximum sound transmission.

### 5. SNAP TIGHT SOFT-SEALING EARTIPS.

Snap-tight, soft-sealing eartips conform to individual ears for an excellent acoustic seal and comfortable fit.

To learn more about Littmann stethoscopes, visit [www.Littmann.com](http://www.Littmann.com) or call the 3M Health Care Helpline at **1-800-228-3957**.

	Cardiology III™ Stethoscope	Master Cardiology™ Stethoscope
Performance (1-10 Scale)	9	10
Chestpiece Design and Technology	Machined Stainless Steel Two-sided Tunable	Polished Stainless Steel Single-sided
Headset Material	Wide Diameter Aerospace Alloy	Wide Diameter Aerospace Alloy
Length	22"/56 cm, 27"/69 cm	22"/56 cm, 27"/69 cm
Total Weight	165, 175 grams	175, 185 grams
Diaphragm Details, Diameter	Two Tunable Diaphragms 1.7"/4,3 cm, 1.3"/3,3 cm	Tunable 1.75"/4,4 cm
Warranty	5 Years	7 Years
Black	3127, 3128, 3131BE <sup>1</sup> , 3128BRS <sup>2</sup> , 3157SM <sup>4</sup> , 3152RBW <sup>5</sup>	2159, 2160, 2161 <sup>1</sup> , 2175 <sup>2</sup> , 2176 <sup>4</sup>
Burgundy	3129	2163
Navy Blue	3130	2164
Hunter Green	3134	2165, 2183 <sup>2</sup>
Plum	3135	2167
Chocolate	3137, 3137CPR <sup>3</sup>	
Gray	3136	
Caribbean Blue	3138	2178
Ceil Blue	3146	
Red	3140	
Raspberry	3148	
Coral	3149	
Sand	3165 <sup>4</sup>	
Dark Olive Green	3166 <sup>4</sup>	2182 <sup>4</sup>
Lavender	3158 <sup>5</sup>	
Turquoise	3159 <sup>4</sup>	
Lemon-Lime	3161 <sup>4</sup>	
Rose Pink	3163 <sup>*</sup>	

Colors shown are approximate. \* Available in the U.S. only.

Special chestpiece/headset finishes are only available in a limited number of tubing colors.



<sup>1</sup> **Black Edition** – black finish chestpiece and eartubes.



<sup>2</sup> **Brass Edition** – brass finish chestpiece and eartubes.



<sup>3</sup> **Copper Edition** – copper finish chestpiece and eartubes.



<sup>4</sup> **Smoke Edition** – smoke finish chestpiece and eartubes.



<sup>5</sup> **Rainbow Edition** – rainbow finish chestpiece and brass finish eartubes.